



Schild Line

3-Ply Disposable Face Mask Made in Germany


Schild Line Face Masks are made out of non-woven and melt blown fabric, providing maximum protection from unwanted micro-particles in the air.

Designed in a way to help users breathe more easily, these masks reduce the feeling of irritating heat or moisture. The inner white non-woven fabric is soft and skin-friendly, minimizing any skin irritation.

Instructions for use:



1. Before applying the mask, wash your hands with soap and water or alcohol-based hand sanitizers.
2. Place the white side of the mask so it completely covers the nose and mouth. Pull both sides of the ear straps and place one behind each ear.
3. The adjustable nose clip must sit at the bridge of your nose and the bottom of the mask must be fixed under your chin. This ensures that it covers your nose and mouth perfectly.
4. Take the mask off by pulling the bands. Do not touch the front blue side of the mask.
5. After removing the mask, immediately throw it in a disposal unit.

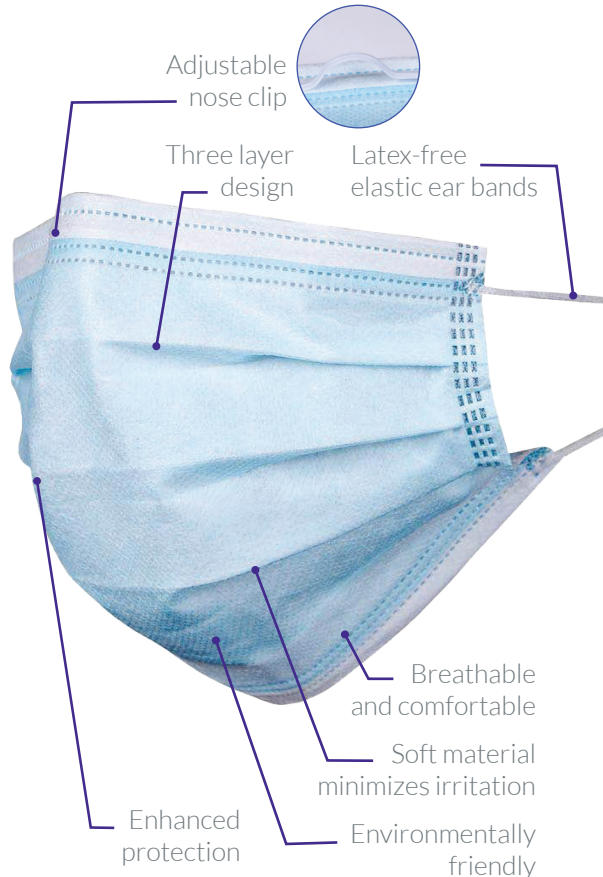



Schild Line

3-Ply Disposable Face Mask Made in Germany

Schild Line Face Masks are designed to provide protection to both mouth and nose. Schild Line is your reliable protection barrier of a high bacterial filtration efficiency, microbial cleanliness and fluid resistance.

FEATURES

- 3-layered masks: Front layer of non-woven blue, PP material, Middle layer of melt blown material of a high bacterial efficiency, Inner layer of a soft non-woven white material.
- Flexible nose clip provides the ultimate fit and attachment to your face.
- Latex-free, elastic ear loops.

SPECIFICATIONS

- Made in Germany
- Colour: Blue
- Size: 175x95mm
- Material tested for: Bacterial Filtration Efficiency, Differential Pressure, Microbial Cleanliness, Resistance to penetration of Synthetic Blood, Cytotoxicity ISO 10993-5.

ORDERING INFORMATION

Cat. No.	Description
6-500-50-2	Face Protection Masks, 50 pcs., Box
6-500-50-2C	Face Protection Masks, 50x50 pcs., Box

